D	GB	F	I	E
---	----	---	---	---

Technische Daten des Gehörschutzes ePRO-STOP nach EN 352-2:2002
Technical data of the hearing protection ePRO-STOP in accordance with EN 352-2:2002
Données techniques de la protection auditive ePRO-STOP selon la norme EN 352-2:2002
Dati tecnici dell’otoprotettore ePRO-STOP conforme a EN 352-2:2002
Datos técnicos de la protección auditiva ePRO-STOP conforme a EN 352-2:2002

	Hz	63	125	250	500	1000	2000	4000	8000	
ePRO-STOP.S	Mf	28,1	32,7	31,4	32,3	32,0	36,4	39,7	33,7	
	Sf	4,9	4,7	3,8	2,9	2,9	2,6	3,8	4,1	
	APV	23,2	28,0	27,6	29,4	29,1	33,8	35,9	29,6	
		SNR: 33 – L: 29, M: 30, H: 33 – W, X, S, V, E <sub>1</sub>								

	Hz	63	125	250	500	1000	2000	4000	8000	
ePRO-STOP.A	Mf	27,8	31,3	31,3	33,1	33,8	36,5	44	43,9	
	Sf	5,9	4,5	4,6	5,5	4,3	3	3,9	4,4	
	APV	21,9	26,8	26,7	27,6	29,5	33,5	40,1	39,5	
		SNR: 34 – L: 28, M: 30, H: 35 – W, S, V, E <sub>1</sub>								

**Bemerkungen**

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#### User instructions for hearing protection ePRO-STOP

With your new, custom-made hearing protection ePRO-STOP you have decided to protect your hearing against damaging or disturbing noise by using custom-made protectors. Your hearing protection device has been manufactured to fit your ear perfectly and is guaranteed to offer you the highest possible wearing comfort. The hearing protector is worn in the ear. It can be inserted with minimal effort, is highly compatible with other protective equipment (e.g. protective glasses) and above all, is appropriate for long stays in noise zones. Properly used, it protects you from hazardous noise. No other use is approved and is therefore not allowed. We hope you enjoy it thoroughly!

#### Prior to using your hearing protection device

Please read these directions for use carefully to familiarize yourself with the proper handling of the hearing protection device. Only if the following suggestions are adhered to can effective pro-tection and the durability of your hearing protectors be guaranteed. Keep these instructions in a safe place.

#### Standard features of your hearing protection device

- Hearing protection earmold (Note color marks: right ear/red, left ear/blue)
- Case
- User instructions

#### Optional additional features and products

- Connecting cord (detachable connection between the right and left hearing protector)

- Safety clip to attach connecting cord to clothes
- Antimicrobial coating
- Metal detection ball

**Inserting and removing your hearing protection device** (see figure 2 to 4)

- With clean hands, take the hearing protection ePRO-STOP out of the packaging.
- Make sure that your entire hearing protectors have not been damaged and are clean. Do not insert a damaged hearing protector in your ear; in case of damage, contact your specialist.
- Note the red and blue color marks on your hearing protectors: red = protector for the right ear; blue = protector for the left ear.
- Take the right protector into your right hand, the left protector into your left hand. Be sure to grasp the protector from the bottom with your thumb and from the top with your index finger (figure 2).
- Take the hearing protector to your ear without twisting it. Angling it from behind, insert it into the auditory canal (figure 3). Be sure to insert the hearing protection device deeply enough and position it comfortably (figure 4).
- To remove the hearing protector grasp the handle with your thumb and index finger and pull the protector out of your ear. Alternatively you can free the hearing protector by lifting it slightly with your thumb and then removing it from your ear with your thumb and index finger (see figure 2).

**WARNING**

- A) Noise impact on hearing without hearing protection
Even a brief exposure to a high noise level while wearing improperly inserted hearing protec-tors or no hearing protection devices at all may damage your hearing.
- B) Protective effect that is too low when the hearing protector is incorrectly inserted in the ear
Only expose yourself to high noise levels while wearing your custom-made hearing protectors after you are familiar with handling the protectors and inserting them properly into the audi-tory canal. Have the correct use shown to you e.g. by the manufacturer or the supplier of the hearing protection earmold or your physician. It may be helpful for beginners to practise the correct technique in front of a mirror.
- C) Handling and daily checks
In the case of obvious defects or alterations of the earmold, do not wear the hearing protec-tion devices in extreme noise levels! Please check your hearing protector for this on a daily basis
- D) Leaking/reduced protective effect
Anatomical changes in the shape of the ear may lead to the deterioration of secure earmold fit, e.g. when there is a change in weight or there have been ear operations. Safe protection is only achieved when a specialist has tested the hearing protection earmold after it is de-livered and then tests it again at regular intervals of at most three years.
- E) Reduced perception
Please consider: While wearing the hearing protection devices, the signals in the surrounding area will also be muffled (e.g. warning signals). In addition, be sure to adhere to the regu-lations regarding the use of hearing protection devices in road traffic/track superstructure/ railway operation.
- F) Risk of injury with improper handling
Do not pull the hearing protectors out of the ear by the connecting cord. Sudden or fast re-moval of the ear plugs out of the auditory canal may damage the ear drum.
- G) Risk of the hearing protector being pulled out.
These ear plugs should not be used where there is a risk that the connecting cord could be caught up during use. We recommend detaching the connecting cord with the connecting fasteners from the hearing protection device. After the connecting cord has been detached, it can only be reattached by inserting it again (note the connecting fasteners in figure 5).

- H) Connecting additional equipment
As a hearing protection device the ePRO-STOP may only be used in connection with additional products that have been type-tested with ePRO-STOP. More information is available from your specialist or the manufacturer.
- I) Skin reaction
The hearing protector is exclusively manufactured from materials that have been proven over the years, tested and that are bio-compatible. If you were nonetheless to notice symptoms, remove the hearing protector immediately and inform your doctor, specialist consultant or supervisors.
- J) For small children there is a swallowing hazard!
The product must be kept out of reach of small children.
- K) Ageing and wear
Custom made earmolds are designed for daily use. However, we recommend having a new spare earmold made after at most 6 years

#### Storage and care

This product may suffer if certain chemical substances are used. Your hearing protectors must be cleaned regularly to avoid a decrease in its protective effectiveness, or the development of skin irritations or other ear problems. Use either special cleaning products offered by the manu-facturer or at least water and soap. Dry the hearing protectors thoroughly before inserting them

again. Do not use chemical solvents, abrasive cleansers or brushes. The attenuation elements must be cleaned professionally. More information is available from your hearing protection pro-fessional or the manufacturer. Store the ePRO-STOP in the case which is included in delivery.

#### ATTENTION

These products may not be altered or modified. If the suggestions in these instructions are not adhered to, the effectiveness of the hearing protection device may be affected considerably. Only by handling the hearing protectors carefully and wearing the hearing protection ePRO-STOP continuously in extreme noise levels, are you protecting your hearing as best as possible. There-fore, we urgently recommend using and maintaining the hearing protection ePRO-STOP accord-ing to these suggestions.

#### CE Declaration of Conformity

AUDIA AKUSTIK GMBH hereby declares that the ePRO-STOP hearing protector conforms to the PPE Directive (EU) 2016/425. The complete declaration of conformity can be found on our homepage at: www.audia-akustik.de (in the ‘Downloads’ section).

Notified body involved in conformity assessment:
PZT GmbH · Bismarckstraße 264 B · 26389 Wilhelmshaven · Germany
Registration number: 1974

Individuell angepaßter Gehörschutz
Custom-made hearing protectors
Protection acoustique sur mesure
Otoprotettore su misura
Protección auditiva a medida

# ePRO-STOP.A ePRO-STOP.S

Wir hören uns.


# AUDIA


Gebrauchsanleitung
User instructions
Mode d’emploi
Istruzioni per l’uso
Instrucciones de uso


**1**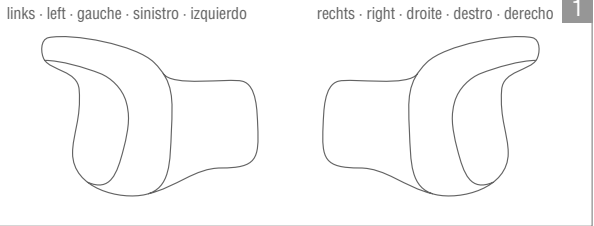

**2**

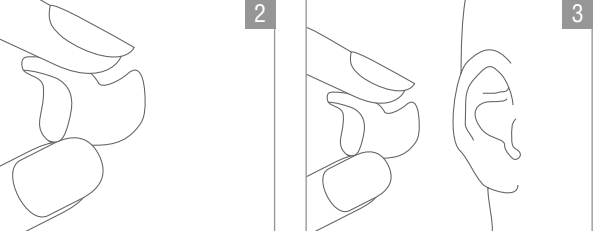**3**

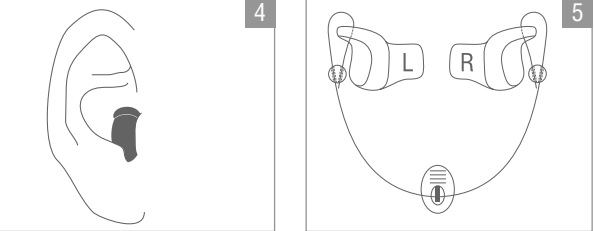**4**

**5**

Zugrunde liegende Normen
Underlying standards
Fabriqué selon les normes de
Norme applicate
Normas en las que se basa
EN 352-2:2002 (DIN EN 352-2:2003) / EN 13819-1:2002
EN 13819-2:2002 / ISO 4869-1:1990 / ISO 4869-2:1994
ISO 4869-3:2007 / ISO 8253-2:2009 / IEC 60268-1:1988